



Hearing Loop Set Up Instructions:

On the DLWP Guest Wifi or at home, download the **Sennheiser MobileConnect App**

Connect to the WIFI name: **Assisted Listening** and enter the password: **Audio1935**
Your phone will prompt you that there is no internet connection but connect anyway.

Open the MobileConnect app. Click on the small blue QR code. Your phone will ask you to allow the app to take a photo. Click "while using this app" or "only this time". The camera will open.

Using your phone's camera, hover over the QR code below and click "Scan this QR code":



A channel called **Assisted Listening** should appear.

Connect your Bluetooth or wired headphones/earphones, personal induction neck loop, or your hearing aids to your phone.

Click on Listen. Sound may start straight away, or you may need to click the Play button. You can adjust the sound using your volume buttons for your own personal level.

Enjoy!

If you are having any issues, please speak to one of the stewarding team.